

## Microsoft Update and PHP Update Monthly Notification

**Summary:** This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES4K, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite™ HD and ES4K, nCare MX/RX, EasyLink™ Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v10.x and up VaultStream systems.

**Products:** EasySuite HD, and ES4K iOR systems and nCare recorders, connected to a network and VaultStream server software, as well as the LiveStream StatusBoard Engine.

**Date:** May 09, 2023

### **Critical Security Bulletins (Obtained from Microsoft):**

**NOTE:** Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

### **Approved Updates:**

May 2023 Updates		
KB Number	Platform	Title
<b>VaultStream v10.x Updates – Windows Server 2012</b>		
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.841.0)
KB5026415	VaultStream 2012	2023-05 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - v5.113
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.835.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.808.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.705.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.659.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.581.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.548.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.495.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.448.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.423.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.305.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.243.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.129.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.47.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.389.44.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2964.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2890.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2827.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2768.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2749.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2703.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2702.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2637.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2593.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2537.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2481.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2422.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2373.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2282.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2184.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2175.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2078.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.2063.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1988.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1944.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1870.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1781.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1755.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1643.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1554.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1429.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1423.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1359.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1309.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1241.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1178.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.1065.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.968.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.899.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.387.885.0)
<b>VaultStream v10.x Updates – Windows Server 2016</b>		
KB5026363	VaultStream 2016	2023-05 Cumulative Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.113
<b>VaultStream v10.4 Updates – Windows Server 2019</b>		
KB5026362	VaultStream 2019	2023-05 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.389.835.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.389.659.0)
KB890830	VaultStream 2019	Windows Malicious Software Removal Tool x64 - v5.113
<b>nCare MX v10.2.3</b>		
KB5026363	nCare MX	2023-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
<b>nCare MX v10.4</b>		
KB5026363	nCare MX	2023-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
<b>ES4K v10.4 ES4K Controller (a.k.a. BlizCon)</b>		
KB5026363	ES4K Controller	2023-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
<b>ES4K v10.4   ES4K Video Manager (a.k.a. nCare RX2)</b>		

KB5026363	ES4K Video Manager	2023-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
<b>ES4K v10.4 LiveStream StatusBoard 2</b>		
KB5026362	LiveStream StatusBoard 2	2023-05 Cumulative Update for Windows 10 Version 1809 for x64-based Systems
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.389.768.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.389.659.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.389.548.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.389.448.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.389.337.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.389.245.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.389.129.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.389.2.0)
KB4052623	LiveStream StatusBoard 2	Update for Microsoft Defender Antivirus antimalware platform - (Version 4.18.2304.8)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.2917.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.2803.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.2702.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.2593.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.2481.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.2357.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.2106.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.2020.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.1929.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.1837.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.1710.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.1619.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.1490.0)

KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.1387.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.1241.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.1166.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.1073.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - KB2267602 (Version 1.387.968.0)
<b>MedPresence Tower v10.4</b>		
KB5026363	MedPresence Tower	2023-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
<b>MAJ-2533</b>		
KB5026362	MAJ-2533	2023-05 Cumulative Update for Windows 10 Version 1809 for x64-based Systems
KB5025229	MAJ-2533	2023-04 Cumulative Update for Windows 10 Version 1809 for x64-based Systems
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.389.915.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.389.852.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.389.817.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.389.581.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.389.387.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.389.187.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.2964.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.2827.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.2749.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.2645.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.2537.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.2422.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.1877.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.1781.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.1423.0)

KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.1326.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.1204.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.1123.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.968.0)
KB2267602	MAJ-2533	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.387.913.0)
KB890830	MAJ-2533	Windows Malicious Software Removal Tool x64 - v5.113
<b>CV Connectivity with KE v3.1 (VaultStream with Interfaces License)</b>		
KB5023696	KE v3.1	2023-03 Cumulative Update for Windows 10 Version 22H2 for x64-based Systems
KB890830	KE v3.1	Windows Malicious Software Removal Tool x64 - v5.111
KB5023765	KE v3.1	2023-03 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB5023790	KE v3.1	2023-03 Servicing Stack Update for Windows Server 2012 R2 for x64-based Systems
KB5023697	KE v3.1	2023-03 Cumulative Update for Windows Server 2016 for x64-based Systems
KB5023788	KE v3.1	2023-03 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB5023702	KE v3.1	2023-03 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems
<b>PHP Updates</b>		
<p>7.4.16, 7.1.33 and 5.6.40 – shipping versions.                      V10.2.2 was Tested with PHP version 7.3.33                      V10.4 was Tested with PHP version 7.4.30                      7.3.33 is available upon request for v10.2.2 VaultStream software only.                      7.4.33 is available upon request for v10.4 VaultStream software only.</p>		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024