

Microsoft Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES4K, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite™ HD and ES4K, nCare MX/RX, EasyLink™ Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v10.x and up VaultStream systems.

Products: EasySuite HD, and ES4K iOR systems and nCare recorders, connected to a network and VaultStream server software, as well as the LiveStream StatusBoard Engine.

Date: October 10, 2023

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

October 2023 Updates		
KB Number	Platform	Title
VaultStream v10.x Updates – Windows Server 2012		
KB890830	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.522.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.522.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.512.0)
KB5031419	VaultStream 2012	2023-10 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - v5.118
KB5031228	VaultStream 2012	2023-10 Security and Quality Rollup for .NET Framework 3.5, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2012 R2 for x64
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.423.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.402.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.374.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.342.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.316.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.291.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.275.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.240.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.233.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.178.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.173.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.136.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.111.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.898.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.399.40.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.2024.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1980.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1971.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1934.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1910.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1880.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1864.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1855.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1833.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1795.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1787.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1729.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1728.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1671.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1634.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1626.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1584.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1572.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1543.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1529.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1506.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1485.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1476.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1450.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1423.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1414.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1374.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1372.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1334.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1321.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1279.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1261.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1239.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1193.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1180.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1146.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1128.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1097.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1065.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1053.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1009.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1007.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.956.0)
VaultStream v10.x Updates – Windows Server 2016		
KB5031000	VaultStream 2016	2023-10 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64
KB5031362	VaultStream 2016	2023-10 Cumulative Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.118
VaultStream v10.4 Updates – Windows Server 2019		
KB5031222	VaultStream 2019	2023-10 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64
KB5031361	VaultStream 2019	2023-10 Cumulative Update for Windows Server 2019 (1809) for x64
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.399.489.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.399.431.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.399.379.0)
KB890830	VaultStream 2019	Windows Malicious Software Removal Tool x64 - v5.118
nCare MX v10.2.3		
KB5031000	nCare MX	2023-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5031362	nCare MX	2023-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB890830	nCare MX	Windows Malicious Software Removal Tool x64 - v5.118
nCare MX v10.4		
KB5031000	nCare MX	2023-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5031362	nCare MX	2023-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB890830	nCare MX	Windows Malicious Software Removal Tool x64 - v5.118
ES4K v10.4 ES4K Controller (a.k.a. BlizCon)		
KB5031000	ES4K Controller	2023-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5031362	ES4K Controller	2023-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB890830	ES4K Controller	Windows Malicious Software Removal Tool x64 - v5.118
ES4K v10.4 ES4K Video Manager (a.k.a. nCare RX2)		
KB5031000	ES4K Video Manager	2023-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5031362	ES4K Video Manager	2023-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB890830	ES4K Video Manager	Windows Malicious Software Removal Tool x64 - v5.118

ES4K v10.4 LiveStream StatusBoard 2		
KB5031222	LiveStream StatusBoard 2	2023-10 Cumulative Update for .NET Framework 3.5, 4.7.2, and 4.8 for Windows 10 Version 1809 for x64
KB5031361	LiveStream StatusBoard 2	2023-10 Cumulative Update for Windows 10 Version 1809 for x64-based systems
MedPresence Tower v10.4		
KB5031000	MedPresence Tower	2023-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5031362	MedPresence Tower	2023-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
MAJ-2533 (UCES-4)		
KB5031222	MAJ-2533	2023-10 Cumulative Update for .NET Framework 3.5, 4.7.2, and 4.8 for Windows 10 Version 1809 for x64
KB5031361	MAJ-2533	2023-10 Cumulative Update for Windows 10 Version 1809 for x64-based systems
CV Connectivity with KE v3.1 (VaultStream with Interfaces License)		
KB5030180	KE v3.1	2023-09 Cumulative Update for .NET Framework 3.5, 4.8 and 4.8.1 for Windows 10 Version 22H2 for x64-based Systems
KB890830	KE v3.1	Windows Malicious Software Removal Tool x64 - v5.117
KB5030211	KE v3.1	2023-09 Cumulative Update for Windows 10 Version 22H2 for x64-based Systems
PHP Updates		
V10.2.2 was Tested with PHP version 7.3.33 V10.4 was Tested with PHP version 7.4.30 7.3.33 is available upon request for v10.2.2 VaultStreams only. 7.4.33 is available upon request for v10.4 VaultStreams only.		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024

Microsoft and Windows are trademarks of the Microsoft group of companies.



Gyrus ACMI, Inc.
 One 9600 Louisiana Ave. North, Brooklyn Park, MN 55445 U.S.A.
www.olympusamerica.com/olympus-systems-integration