

Microsoft Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES4K, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite™ ES4K and ESIP, nCare MX and 4K, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v10.x and up VaultStream systems.

Products: EasySuite™ ES4K and ESIP iOR systems and nCare MX and 4K recorders, connected to a network and VaultStream server software.

Date: April 27, 2026

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggest they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

April 2026 Updates		
Platform	KB	Title
VaultStream v10.6 Updates – Windows Server 2016		
VaultStream 2016	KB5082411	2026-04 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64 (KB5082411)
VaultStream 2016	KB5082089	2026-04 Servicing Stack Update for Windows Server 2016 for x64-based Systems (KB5082089)
VaultStream 2016	KB5082198	Cumulative Update for Windows Server 2016 for x64-based Systems (KB5082198)
Vault Stream	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (version 1.449.228.0) - Current Channel (Broad)
VaultStream 2016	KB890830	Windows Malicious Software Removal Tool x64 -v5.140
VaultStream v10.6 Updates – Windows Server 2019		
VaultStream 2019	KB5082123	2026-04 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems (KB5082123)
VaultStream 2019	KB5084066	2026-04 Cumulative Update for .NET Frameworks 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64 (KB5084066)
VaultStream 2019	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.449.228.0) - Current Channel (Broad)
VaultStream 2019	KB890830	Windows Malicious Software Removal Tool x64 –v5.140 (KB890830)
VaultStream v10.7 Updates – Windows Server 2019		
VaultStream 2019	KB5082123	2026-04 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems (KB5082123)
VaultStream 2019	KB5084066	2026-04 Cumulative Update for .NET Frameworks 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64 (KB5084066)
VaultStream 2019	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.449.224.0) - Current Channel (Broad)
VaultStream 2019	KB890830	Windows Malicious Software Removal Tool x64 -v5.139
VaultStream v11.5 Updates – Windows Server 2019		
VaultStream 2019	KB5082142	2026-04 Cumulative Update for Microsoft server operating system version 21H2 for x64-based Systems (KB5082142)
VaultStream 2019	KB5084071	2026-04 Cumulative Update for .NET Framework 3.5, 4.8 and 4.8.1 for Microsoft server operating system 21H2 for x64 (KB5084071)
VaultStream 2019	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus –

		KB2267602 (Version 1.449.222.0) - Current Channel (Broad)
VaultStream 2019	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.447.149.0) - Current Channel (Broad)
VaultStream 2019	KB890830	Windows Malicious Software Removal Tool x64 -v5.140
VaultStream v11.5 Updates – Windows Server 2022		
VaultStream 2022	KB5082142	2026-04 Cumulative Update for Microsoft server operating system version 21H2 for x64-based Systems (KB5082142)
VaultStream 2022	KB5084071	2026-04 Cumulative Update for .NET Framework 3.5, 4.8 and 4.8.1 for Microsoft server operating system 21H2 for x64 (KB5084071)
VaultStream 2022	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.449.222.0) - Current Channel (Broad)
VaultStream 2022	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.447.149.0) - Current Channel (Broad)
VaultStream 2022	KB890830	Windows Malicious Software Removal Tool x64 -v5.140
nCare MX v10.2.3		
nCare MX	KB5082198	2026-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems (KB5082198)
nCareMX	KB5082089	2026-04 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems (KB5082089)
nCareMX	KB5082411	2026-04 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 (KB5082411)
nCare MX	KB890830	Windows Malicious Software Removal Tool x64 -v5.140
nCare MX v10.4		
nCare MX	KB5082198	2026-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems (KB5082198)
nCare MX	KB5082089	2026-04 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems (KB5082089)
nCare MX	KB5082411	2026-04 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 (KB5082411)
nCare MX	KB890830	Windows Malicious Software removal Tool x64 -v5.140
nCare 4K v11.3		
nCare 4K	KB5082200	2026-04 Cumulative Update for Windows 10 Version 21H2 for x64-based Systems (KB5082200)
nCare4K	KB5084067	2026-04 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 (KB5084067)
nCare 4K	KB890830	Windows Malicious Software Removal Tool x64 -v5.140
nCare 4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.449.230.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.449.222.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.449.199.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.449.183.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.449.170.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft defender Antivirus – KB2267602 (Version 1.449.152.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.449.136.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus –



		KB2267601 (Version 1.449.120.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft defender Antivirus – KB2267602 (Version 1.449.106.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.449.89.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.449.48.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.449.34.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.447.236.0)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus- KB2267602 (Version 1.447.224.0)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus- KB2267602 (Version 1.447.190.0)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus- KB2267602 (Version 1.447.176.0)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus- KB2267602 (Version 1.447.163.0)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus- KB2267602 (Version 1.447.149.0)
nCare4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus- KB2267602 (Version 1.447.137.0)
nCare4k	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus- KB2267602 (Version 1.447.125.0)
nCare 4K	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus – KB2267602 (Version 1.447.224.0)
nCare4K	KB4052623	Update for Microsoft Defender Antivirus antimalware Platform Antivirus- KB4052623 (Version 4.18.26030.3011)
ES4K v10.4 ES4K Controller (a.k.a. BlizCon)		
nCare MX	KB5082198	2026-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems (KB5082198)
nCare MX	KB5082411	2026-04 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 (KB5082411)
nCare MX	KB5082089	2026-04 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems (KB5082089)
NCare MX	KB890830	Windows Malicious Software Removal Tool x64 -v5.140
ES4K v10.4 ES4K Video Manager (a.k.a. nCare RX2)		
nCare MX	KB5082198	2026-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems (KB5082198)
nCare MX	KB5082411	2026-04 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 (KB5082411)
nCare MX	KB5082089	2026-04 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems (KB5082089)
nCare MX	KB890830	Windows Malicious Software Removal Tool x64 -v5.140
ES-IP iOR System v11.4		
ES-IP iOR System	KB5082200	2026-04 Cumulative Update for Windows 10 Version 21H2 for x64-based Systems (KB5082200)
ES-IP iOR System	KB5084067	2026-04 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 (KB5084067)
ES-IP iOR System	KB890830	Windows Malicious Software Removal Tool x64 -v5.140
CV Connectivity with KE v3.4.0 (VaultStream with Interfaces License) - Windows 11 (24H2)		



KE v3.4.0	KB5079473	2026-03 Security Update (KB5079473) (26200.8037)
KE v3.4.0	KB890830	Windows Malicious Software Removal Tool x64 - v5.140 (KB890830)
CV Connectivity with KE v3.1.1 (VaultStream with Interfaces License) - Windows Server 2016		
KE v3.1.1	KB5078938	2026-03 Cumulative Update for Windows Server 2016 for x64-based Systems (KB5078938)
KE v3.1.1	KB890830	Windows Malicious Software Removal Tool x64 - v5.140 (KB890830)
CV Connectivity with KE v3.4.0 (VaultStream with Interfaces License) - Windows Server 2019		
KE v3.4.0	KB5078752	2026-03 Cumulative Update for Windows Server 2019 (1809) for x64 -based systems (KB5078752)
KE v3.4.0	KB890830	Windows Malicious Software Removal Tool x64 - v5.14 (KB890830)
CV Connectivity with KE v3.4.0 (VaultStream with Interfaces License) - Windows Server 2022		
KE v3.4.0	KB5078766	2026-03 Cumulative Update for Windows Server 2019 (1809) for x64 -based systems (KB5078766)
KE v3.4.0	KB890830	Windows Malicious Software Removal Tool x64 - v5.140 (KB890830)
CV Connectivity with KE v3.4.0 (VaultStream with Interfaces License) - Windows Server 2025		
KE v3.4.0	KB5078750	2026-03 Security Update (KB5078750) (26100.32522)
KE v3.4.0	KB890830	Windows Malicious Software Removal Tool x64 - v5.140 (KB890830)
PHP Updates		
<p>7.4.16, 7.1.33 and 5.6.40 – shipping versions. V10.2.2 was Tested with PHP version 7.3.33 V10.4 was Tested with PHP version 7.4.30 7.3.33 is available upon request for v10.2.2 VaultStreams only. 7.4.33 is available upon request for v10.4 VaultStreams only.</p>		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024



Gyrus ACMI, Inc.
 One 9600 Louisiana Ave. North, Brooklyn Park, MN 55445 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Microsoft and Windows are trademarks of the Microsoft group of companies.